## PL

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## EN

### IMPORTANT SAFETY AND USE WARNINGS

Intended use: The nasal aspirator allows for the safe and hygienic removal of nasal secretions from a child.

#### Instructions for use:

Squeeze the aspirator pump, then gently place the nozzle next to one nostril of the child without inserting it into the nose. Gently close the other nostril with your finger. Slowly release the pressure on the pump, which will allow for the effective and hygienic removal of nasal secretions from the child.

#### Safety recommendations:

**WARNING:** Never insert the aspirator nozzle into the nose of a child to avoid the risk of damage. The aspirator is intended for use under adult supervision only. Do not use the aspirator if there is visible damage to the components.

#### Cleaning and maintenance:

After each use, remove the nozzle and wash it thoroughly in warm water. Remove excess water, then leave the components in the opening upwards to dry completely. To sterilize, place the aspirator in a pot of boiling water for 5 minutes, making sure there is enough water to prevent damage to the components.

**WARNING:** Do not sterilize in a microwave or steam sterilizer - this may damage the

product. Retain these warnings and instructions for future reference.

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