**PL**

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## EN

Safety rules:

Please read the instructions below carefully before using the product.

- Use the product only for its intended purpose.
- The nipple shield is for personal use and must not be used by more than one person to avoid cross-contamination.
- The nipple shield can be used multiple times by the same person provided that hygiene rules are followed. Please disinfect or boil it at least once a day.
- We recommend consulting a doctor, midwife or lactation consultant.
- Before each use, please check the nipple shield for damage and replace it if you notice the slightest cracks or bite marks - there is a risk of pieces of the shield breaking off and being swallowed by your baby.
- After cleaning, the nipple shield should be stored in a dry place and protected from light. We recommend a protective box for storage. For storage, place the washed and dried nipple shield in the lid of the box with the open side of the shield facing upwards and close the box.
- The nipple shield and the protective box are not toys and should not be in the hands of children. Please keep the manufacturer's address, article number and batch number for any questions.

Made in Germany.

Instructions:

1 Before putting on the nipple shield, wash your hands thoroughly and wipe your breast with a damp cloth. Do not use soap or alcohol, as this can dry out the skin and crack the nipples. To improve the adhesion of the nipple shield to the breast, you can moisten the inside of the nipple shield with water or food.

2 Fold the base of the nipple shield outwards (towards the tip) and place the nipple shield in the middle of the nipple.

3 Then attach the base of the nipple shield so that it lies comfortably against the skin.

4 The position of the nipple shield depends on the intended breastfeeding position. Please make sure that the cutout in the nipple shield is positioned towards the baby's nose. This allows direct contact between the baby's nose and the breast. After feeding, please wipe the breast with a damp cloth to prevent milk residue from drying and microorganisms from developing. If you have any problems using the nipple shield, please contact your lactation consultant, midwife or doctor.

When selecting the correct size of the nipple shield, the physical conditions of the mother and child should be taken into account. The nipple should fill the entire empty space in the nipple shield (S: 16 mm; M: 20 mm; L: 24 mm), but the nipple shield should not press against the breast. The baby's mouth should cover the entire hood and be close to the mother's breast to allow for optimal milk flow.

Cleaning the nipple shield: Important!

- Please use only drinking quality water for cleaning the nipple shield as described here. • The nipple shield should be washed immediately after use to prevent food residues from drying on and bacteria from growing.
- The nipple shield can be used multiple times by the same person, provided that hygiene rules are followed. Please disinfect or boil it at least once a day. Before first use and after each subsequent use, the nipple shield should be washed thoroughly with water and a mild dishwashing liquid immediately. We recommend NUK Bottle and Teat Cleaner. After washing, please rinse the nipple shield with clean water and dry in a hygienic environment (e.g. in the NUK Storage Box or on a clean, dry, lint-free cloth). When washing in the dishwasher, damage to the material cannot be completely ruled out. We therefore do not recommend washing in the dishwasher.
- Additionally, disinfect or boil the nipple shield at least once a day.
- We recommend the NUK Steriliser for disinfection.
- When boiling, ensure that there is sufficient water to prevent damage to the nipple shield.

The nipple shield must be completely covered with water to prevent damage.

The boiling time is 3 minutes. Discolouration may occur when boiling or sterilising together with

other products. This does not affect the functionality of the product. If the nipple shield is not used after cleaning, please store it in the enclosed box after drying to protect it from external factors. Please also clean the NUK storage box regularly. For this purpose, you can use NUK bottle and teat cleaner and a clean, dry, lint-free cloth.

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