PL

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EN

This instruction should be kept for future reference, as it contains important information.

Cleaning Before using the breast pump, wash your hands with soap and water, then dry them thoroughly.

Sterilize the silicone manual breast pump using the provided lid by placing it in the microwave or in a cold water sterilizing bath, using steam or UV sterilization.

The milk storage bag included in the packaging is single-use, sterile, and ready to use.

How to use the self-sterilizing lid Wash the breast pump with detergent and then rinse it thoroughly with clean water.

Fill the breast pump with 30 ml of water.

Place the lid on the breast pump.

Place it in the microwave for 4 minutes (750-1000 W).

Always leave it to cool for at least 2 minutes before removing it from the microwave.

Instructions Always store the silicone breast pump in a cool, dry place, away from heat and moisture, and out of direct sunlight.

Keep the silicone breast pump out of the reach of infants and children.

Inside, there are important instructions. Please read and keep them for future reference. Contact details and instructions can be found at www.tommeetippee.com.

How to attach the silicone breast pump Gently squeeze the bottom of the breast pump.

Gently place the bowl onto your breast, positioning your nipple and areola inside.

The nipple should fit snugly around the narrow neck of the device.

Gently squeeze the breast pump a few times or massage the breast until milk starts to flow.

Tips Only use a domestic microwave. Do not place it in conventional ovens or grills. Do not use the microwave for sterilization in microwaves over 1200 W.

Do not use the lid with stainless steel inserts.

Do not use with commercial devices.

Avoid any sharp metal parts inside the microwave. Follow the instructions of the microwave device.

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