PL

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EN

### IMPORTANT WARNINGS

Keep this manual for future reference as it contains important information. This manual can also be found at [tommeetippee.com](http://tommeetippee.com).

Mom, we know you have a lot on your plate, but may we suggest you familiarize yourself with your breast pump before you need to use it? This will make the experience less stressful, and soon you'll become an expert (just like with many other new things).

#### Get to Know Your Breast Pump

- A. Funnel – The trumpet-shaped part that supports the cup during use.
- B. Body – The curved part that connects the funnel and cup to the bottle and attaches to the handle.
- C. One-way Valve (peach-colored) – A small one-way valve that allows milk to flow into the bottle but prevents it from flowing back.
- D. Connector – The stick-shaped element that connects the handle to the diaphragm.

E. Diaphragm – A small, transparent, cup-shaped part that creates the suction.

F. Handle and Cap – The curved handle used for pumping. It locks onto the body and diaphragm using the connector.

#### WARNINGS

Parts not currently in use should be stored in a place out of reach of children.

The device should not be cleaned with solvents or aggressive chemicals.

During pumping, do not place fingers between the handle and the body of the pump or the bottle, as it could cause them to become stuck.

Other than cleaning and sterilizing, no further maintenance is required by the user.

Do not leave the device in direct sunlight.

Do not allow the product to come into contact with oil-based food products (e.g., tomato sauces with oil) as discoloration will occur.

This product is intended for personal hygiene, so it cannot be returned once the packaging has been opened.

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